

FREE ACCESS™ II

CHEEK AND LIP RETRACTOR

Cheek and lip retraction without a second pair of hands.

INFORMATION FOR USE

AVAILABLE IN TWO SIZES:

FREE ACCESS II – Small Item # 21-303100S

FREE ACCESS II – Large Item # 21-303100L

FEATURES

- Perfect companion to in-office bleaching.
- Transparent material is ideal for clinical photography
- Adjustable tensility for maximum patient comfort.
- Unique intraoral extensions retract buccinator muscles
- Safe for autoclave and dry heat sterilization
- Easy to use

A VITAL CHAIRSIDE COMPANION FOR A VARIETY OF DENTAL PROCEDURES

- **Bleaching, air abrasion, microetching, surgical and laser procedures**
Increases access to intraoral areas.
- **Impression taking, bite registration, denture try-in**
Helps guide the patient into maximum intercuspation (MI).
- **Sealant applications, composite resin bonding, fluoride application**
Shortens intraoral working time in difficult to reach areas.
- **Oral prophylaxis, periodontal charting, scaling, root planning**
Prevents operator muscle fatigue due to continuous cheek retraction.
- **Clinical photography**
Requires no assistance and transparent material is ideal for clinical photography.

INSTRUCTIONS FOR USE

1. To facilitate patient comfort, apply lip balm or petroleum jelly to the patient's lips prior to inserting FREE ACCESS™ II.
2. FREE ACCESS™ II comes fully assembled. Hold both ends and squeeze the retractors together gently bending the stainless steel bow. The bow is positioned at the patient's chin and provides necessary tension to hold the device in place without manual assistance. (In some cases, it may be more convenient to position the bow over the bridge of the patient's nose.)
3. Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor.

4. The pre-adjusted tension of the stainless steel bow should be comfortable for most patients. However, if an adjustment is necessary, disassemble the device by rotating the plastic interlocking pivot ring at the end of each cheek retractor, make the following adjustments, and reassemble.
 - To reduce tension of the bow, gently bend the bow to increase its curve.
 - To increase tension of the bow, gently bend the bow to gain a straighter curve.
5. To remove FREE ACCESS™ II from the mouth, carefully grasp the left and right cheek retractors and squeeze together to release the tension on the bow then gently remove the device from the mouth.

INSTRUCTIONS FOR CLEANING AND STERILIZATION

- **General cleaning before sterilization**

The product disassembles into three parts for cleaning. FREE ACCESS™ II may be autoclaved or dry heat sterilized. Using a mild cleanser and soft brush or sponge, clean and rinse FREE ACCESS™ II of any debris, blood or saliva before sterilization.
- **Dry heat sterilization**

FREE ACCESS™ II may be sterilized by dry heat method at temperatures up to 400°F (204°C). When using dry heat sterilization, always load other instruments in the tray first, then place FREE ACCESS™ II on top of the other instruments. **DO NOT PLACE ANY OTHER INSTRUMENTS ON TOP OF FREE ACCESS II** as this may cause the material to crack.
- **Autoclave**

FREE ACCESS™ II may be sterilized by autoclaving at temperatures up to 275°F (135°C). Follow the same guidelines for loading as in dry heat sterilization. Chemical sterilization is not recommended as it will cause discoloration and damage FREE ACCESS™ II.

To order

FREE ACCESS™ II contact your local dealer or call 888-JMORITA (566-7482).