

# Tray Adhesive for A-Silicones



Tray Adhesive for A-Silicones ensures perfect adhesion between addition silicone-based impression materials (VPS) and minimally or non-retentive impression trays (metal, plastic or resin). Please check compatibility with your plastic tray.

## APPLICATION

1. Use the brush to apply a fine, regular layer of Tray Adhesive for A-Silicones onto the internal surface of the impression tray before applying the impression material.  
The impression tray must be perfectly clean, grease-free and dry.
2. Allow to dry for 3 minutes, you then have 30 minutes at maximum to apply the silicone.

## PRECAUTIONS FOR USE

- Do not swallow
- Highly inflammable
- Do not breathe the vapour
- Avoid contact with skin and eyes

## SIDE EFFECTS

There are no known systemic side effects to date, however,

- do not use in cases where known allergies to the components exist
- in some cases individual contact allergies may exist

## COMPOSITION

Silicone resin, solvent (dichloromethane, toluene).

## STORAGE

Store at room temperature ( $\leq 77$  °F/25 °C).

Do not use after expiration date.

Close the bottle tightly after use.

## DISPOSAL

Please dispose in accordance with local regulations.

## PACKAGING

1 bottle @ 10 ml liquid

REF 999394

**Caution:** Federal Law restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

Exclusively Distributed in the United States and Canada by:

Zenith Brands Division of Foremost Dental Mfg. Co., Inc.

Telephone number: 1-800-662-6383

DMG Chemisch-Pharmazeutische Fabrik GmbH  
Elbgaustraße 248 • 22547 Hamburg • Germany  
Tel: +49 40-84 00 60 • Fax: +49 40-84 00 62 22  
info@dmg-hamburg.de • www.dmg-hamburg.de



**Keep away from children! For dental use only!**