


**NEVER AGAIN WRAP POLISHING STRIPS OR SAWS
AROUND YOUR FINGERS! STOP TRAUMATIZING
SOFT TISSUE...**

**Introducing the new
LASCHAL/WEINBERG FORCEPS**


- Reduces incidence of soft tissue trauma
- Provides comfort for you and your patient
- Easy to use
- Greater economics

PCF-LW/L \$285.00

*Prongs are perpendicular to handle -
more efficient for use with
polishing strips.*

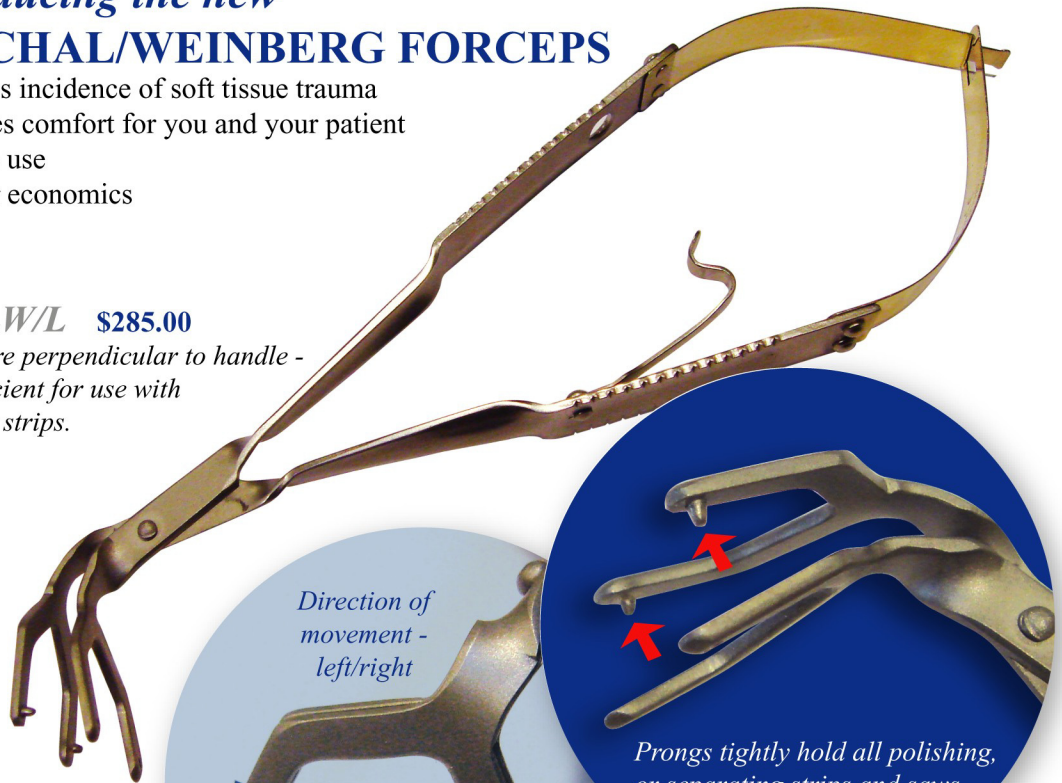

*Direction of
movement -
left/right*

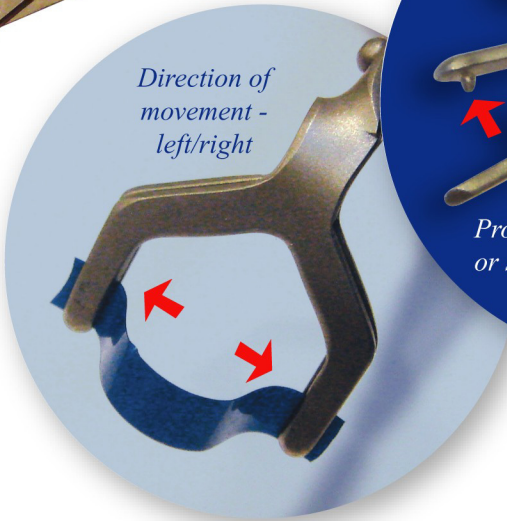

*Prongs tightly hold all polishing,
or separating strips and saws.*


*Loop polishing strip
to access all restored
surfaces or with
articulating paper to
uncover excessive
contact during try-ins.*

PCF-LW/L-ST \$285.00

*Prongs are in line with handle -
more efficient for use with
separating strips or saws.*



*Direction of movement -
forward/backward*

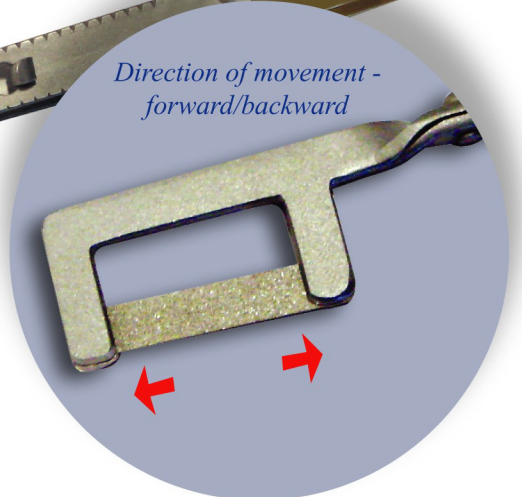


LIMITED TIME SPECIAL OFFER:

Purchase both PCF-LW/L and PCF-LW/L-ST \$395.00


LaschalSurgical, Inc.

HOW TO PREPARE YOUR DOUBLE PRONGED FORCEPS FOR MAXIMAL EFFICIENCY

LASCHAL/WEINBERG FORCEPS PCF-LW/L

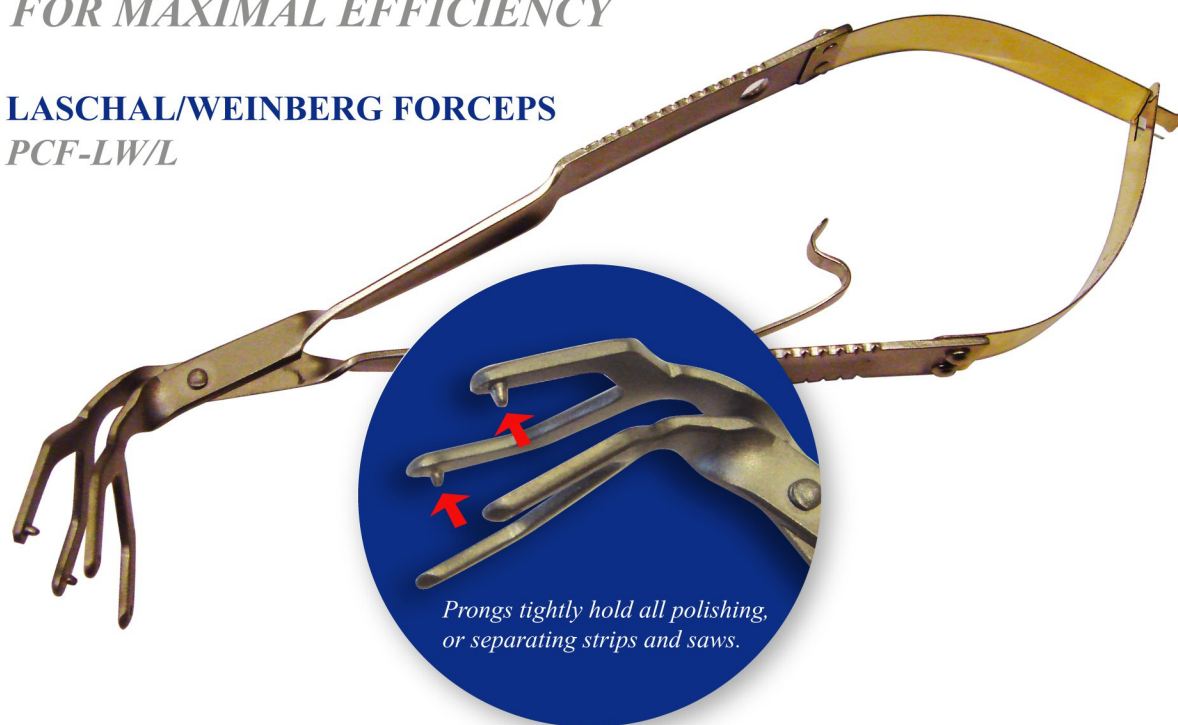


A - For separating strips and saws

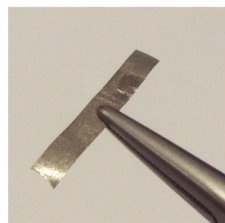

1 Cut a 1" length of strip and secure midway with a hemostat or pliers



2 Place strip between prongs

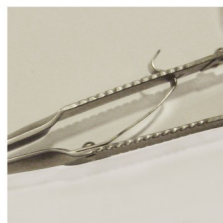


3 Close instrument so that the lock engages




4 Notice that the pins have not yet penetrated strip on either side



5 Use hemostat or pliers to (gently) clamp down on one prong and make sure the pin has penetrated the strip on that side only

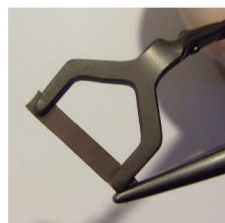

6 Use Hemostat to draw the strip taut on the opposite (unpenetrated) side

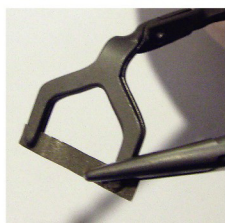

7 Use hemostat to clamp down and penetrate opposite side



8 Notice fully closed, locked and penetrated strip (on both sides)



9 Trim overhanging strip on both sides



10 The strip is now (very) tightly held and ready for use

B - For polishing strips and articulating paper

Loop strip or paper (with greater or lesser loop, as desired) as shown

1 Once positioned, closing and locking the forceps (as in **3** above) usually completes penetration - if not, finger pressure alone will complete penetration

2 Trim as in (9) above

