

O-RING REPLACEMENT - MICROETCHER II/IIA

1. Unscrew and remove nozzle A from coupling/body B.
2. Remove o-ring size #1 and #3 from coupling B.
3. Using pliers, unscrew and remove carbide mounting C from nozzle A. Excessive pressure may cause damage.
4. remove o-ring size #2 from nozzle A.
5. Slide out collar D.
6. Remove o-ring size #1 from inside collar D.
7. Install o-ring size #1 and insert o-ring size #3 into coupling B.
8. Install o-ring size #1 inside collar D.
9. Slide in collar D and o-ring size #2 into nozzle A.
10. Screw and tighten carbide mounting C into nozzle A.
11. Re-install nozzle A to coupling/body B.

