

# Prelude

- Compatible with all composites and compomers.
- **Eugenol** will inhibit set. Phosphoric etchant will remove residual eugenol.
- **Avoid saliva and blood contamination.**
- **Shake Adhesive well prior to first use** to redisperse fillers and gently after extended storage.
- **Point nozzle straight down** when dispensing. Release pressure, set up-right and cap tightly.
- **Self-cure/dual-cure** composites require Prelude Dual/Self-Cure Link.
- **Store <85°F** and out of sunlight. Refrigeration may extend shelf life.
- **Caution:** Unpolymerized monomers may cause skin irritation. Wash thoroughly with soap and water in case of contact.

90987 Rev A

WWW.DANVILLEMATERIALS.COM  
 800.827.7940 925.973.0710 FAX 925.973.0764  
 SAN RAMON, CALIFORNIA 94583 USA  
 3420 FOSTORIA WAY SUITE A-200

**Danville**  
 ENGINEERING & MATERIALS



Reorder Numbers:

Ref # 90990	Prelude Self-Etch Kit
Ref # 90994	Prelude Total-Etch Kit
Ref # 91024	Prelude Primer/Adhesive 2/pk 5ml ea
Ref # 90971	Prelude Primer 5ml
Ref # 90973	Prelude Adhesive 5ml
Ref # 90975	Prelude Dual/Self-Cure Link 5ml
Ref # 87001	MicroPrime Desensitizer 10ml
Ref # 89463	SureEtch Gel 5gm

# Prelude

ADHESIVE SYSTEM



**Prelude Self-Etch Kit**  
 5ml each Prelude Primer, Adhesive & Dual/Self-Cure Link,  
 40 Microbrushes & 10 Mixing Wells

**Prelude Total-Etch Kit**  
 5ml each Prelude Adhesive & MicroPrime, 5gm SureEtch,  
 40 Microbrushes & 10 Mixing Wells

Danville



# Prelude INSTRUCTIONS

Prelude is a nanohybrid, light cured, self-etch or total-etch dental adhesive ideal for bonding composite to dentin, enamel and porcelain. It's low film thickness does not interfere with the fit of indirect restorations.

## SELF-ETCH



Apply **Primer** to enamel & dentin. **Scrub for 10 sec** with a MicroBrush®. Do not rinse.




Use an air syringe to gently **evaporate the solvent**.



Apply **Adhesive** with a new MicroBrush®. **Scrub for 10 sec**.



**Evaporate solvent** with gentle air stream. Increase air pressure to **thin adhesive**.

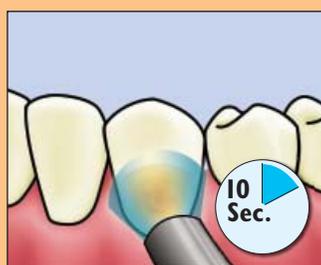


**Light cure 10 sec.**

**For direct restorations:**  
Place 1-2 mm layer of composite (flowable preferred) or compomer restorative and light cure.


## TOTAL-ETCH



**Etch surface for 10 sec** with SureEtch phosphoric etchant. **Do not scrub. Rinse well.**

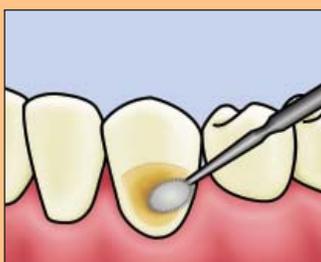


**Do not blow dry. Blot excess water** with a new MicroBrush®, foam pellet or large paper point.

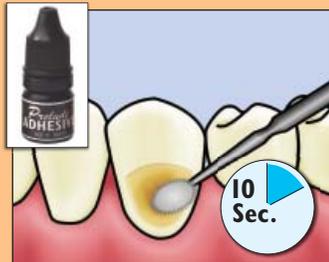

Apply **Adhesive** with a new MicroBrush®. **Scrub for 10 sec**.



**Evaporate solvent** with gentle air stream. Increase air pressure to **thin adhesive**.

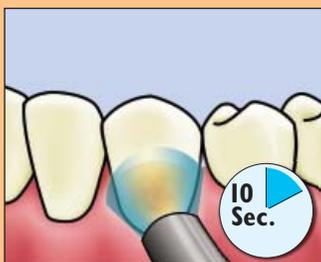

**Light cure 10 sec.**

**For direct restorations:**  
Place 1-2 mm layer of composite (flowable preferred) or compomer restorative and light cure.

## DUAL/Self-CURE LINK FOR BONDING SELF-CURE & DUAL-CURE COMPOSITES

- Apply Link onto light cured Adhesive with a MicroBrush®.
- Evaporate solvent with air syringe.
- Apply self-cure or dual-cure composite.

