



Danville

INNOVATIVE DENTAL PRODUCTS

3420 Fostoria Way Suite A-200
San Ramon, California 94583
Phone 925/973-0710
Phone 800/827-7940
Fax 925/973-0764

From the pioneers in dental air abrasion.

PrepStart™ H₂O

**INSTRUCTIONS FOR RETRO-FITTING AN EXISTING PREPSTART
FOOT PEDAL FOR USE WITH THE H₂O UNIT**

1. Items required:
 - 1.1. Scissors
2. Slide the black cable tie away from the Red & Blue connectors approximately 6 inches.
3. Cut the clear tubing approximately 1 inch in front of the black cable tie and set aside.
4. Slide one of the white plastic tube restraints onto the clear tube and install the brass parallel tee fitting into the clear tubing, push the barb portion of the fitting all the way into the inside diameter of the tube, the parallel portion of the tee should face away from the foot pedal.
5. While holding the brass tee fitting, pull back slightly on the clear tubing with your other hand and slide the white plastic tube restraint forward until it is flush against the brass fitting.
6. Place the other white plastic tube restraint onto the clear tubing cut earlier and set aside, install the tubing onto the remaining barb of the brass tee and repeat step 5.
7. Install the black cable tie approximately a couple of inches from the blue and red connectors.
8. Your foot pedal is now ready to connect to your PrepStart H₂O unit.

