


## Sand-Trap™

The SAND-TRAP is a patented clear plastic device designed to confine and suction away aluminum oxide particulates during inter-oral or chairside sandblasting. As demonstrated in fig. 1, a standard high volume suction tip is inserted close to the side hole, while the tip of the Microetcher fits into the small hole snugly.

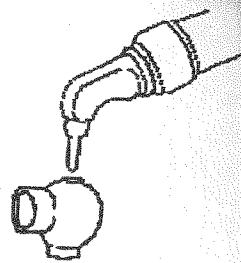

### HELPFUL HINTS:

The SAND-TRAP can be trimmed at the base with scissors for better adaptation, angulation, or simply to bring the Microetcher tip closer to the surface to be sandblasted.

Place the SAND-TRAP on your Microetcher and try it on a penny to make sure the whole assembly is working properly. Do NOT place the sandblasting tip less than 3 millimeters from the blasting surface, otherwise back pressures may force some sand particles to shoot out of the container.

The SAND-TRAP may be cold sterilized for multiple use.

### CAUTION:

While the SAND-TRAP will catch most of the particles, it is not 100% effective. Therefore, eye protection is a must for the doctor, assistant, and patient during use.

Part number 22006



0044



**Danville**

3420 Fostoria Way, Ste A-200, San Ramon, CA 94583

Phone: (800) 827-7940 Fax: (925) 973-0764