


Now available with reflective backing!
Try our new Reflective DryTips®

 Video and instructions for use available
 at www.microbrush.com/drytips

Ultimate Saliva Control

Far outlasts cotton rolls and other absorbent pads, and
 doesn't lose moisture when handled.

The Dry Tips advantage over traditional cotton rolls

- Complete coverage of parotid orifice and entire buccal mucosa with polyethylene film for cutting protection.
- Facilitates work far back in oral cavity.
- Flexible; accommodates to cheek movements.
- Comfortably stays in place, yet is easily removed with water spray.
- Reflective coating for increased visibility and to aid in retraction.



**NEW! Reflective coating
 now available.**

Ideal for:

- Impressions
- Occlusal Adjusting
- Orthodontics
- Paint-on techniques
- Composites
- Cementation
- Fissure sealing
- Fluoride applications

Order today through your preferred dealer.
For questions, call Microbrush® International
at 1 (866) 866-8698.

DryTips Ordering Information:

ITEM NUMBER	SIZE	QUANTITY	COLOR
291542	Large	50	White
291543	Small	50	White
291726	Large	50	Silver
291727	Small	50	Silver



**Cotton Rolls Fray,
DryTips Stay!**

**Moisture control with
patient comfort in mind.**

Ultimate Saliva Control

Far outlasts cotton rolls and other absorbent pads, and doesn't lose moisture when handled.


As easy as one, two, three:

Follow the simple instructions below for easy, ultimate saliva control from a brand you trust.

Instructions:


Step 1

- Insert DryTip arrow side to the back of the mouth so the mesh is touching the inside of the cheek.

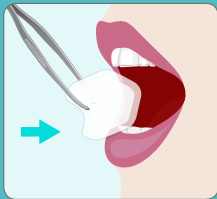


Step 2

- Simply apply water to remove.

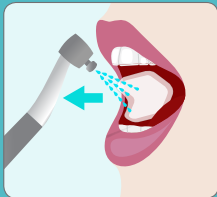


Step 3

- Remove with ease when fully saturated.

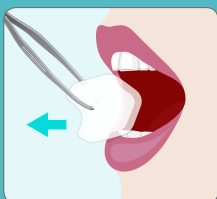

Slim Profile: To facilitate work far back in the oral cavity without interference.

Complete Comfort: Designed to fit seamlessly in the mouth with flexible edges that accommodate to cheek movements.

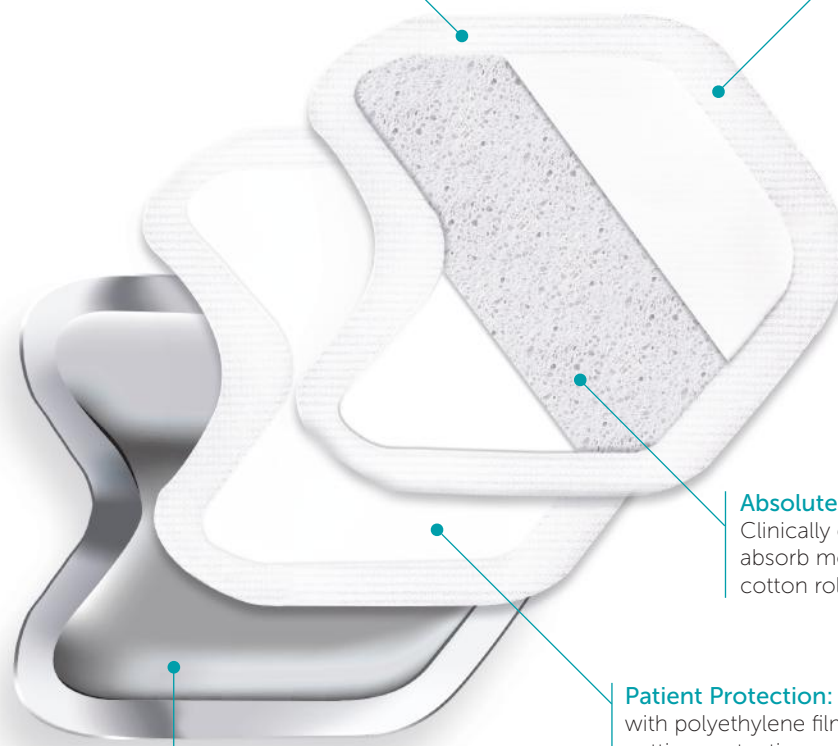

Absolute Absorption: Clinically engineered to absorb more moisture than cotton rolls and won't fray.

Patient Protection: Coated with polyethylene film for cutting protection against sharp instruments.

Optional Reflective Backing

Increases visibility in the oral cavity during procedures.

Order today through your preferred dealer.

For questions, call Microbrush® International at 1 (866) 866-8698.