





User's Instructions for CP5-O <with Fiber Optic> Coupling

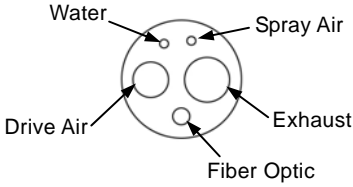

* Before use check the following items:
Does the connection end of the tube match the coupling? This coupling cannot be connected to a tube that does not have light capability. This coupling cannot be connected to a 4-hole tube.0


1. Connect Coupling

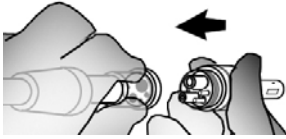

Line up the projections on the coupling with the indentations in the tube and then tighten up the cover nut.

⚠ CAUTION

- Before connection, make sure the tube connector is clean and free of debris.
- The light will lose its brightness if the end of the fiber optic cable is scratched or otherwise damaged by being dropped for instance.

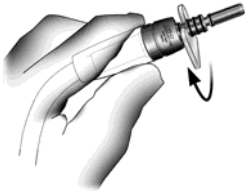

Tighten securely with the wrench provided.

⚠ CAUTION

- Air or water may leak if the coupling is not tightened up enough.

* Before first using the coupling or if the handpiece-coupling connection gets stiff, take the nozzle off the AR Spray can and lightly spray the coupling's o-rings.

2. Cleaning

Wipe the outside cover with ethanol.

⚠ CAUTION

- Coupling must NOT be autoclaved or cleaned ultrasonically.
- The coupling could be damaged if it is soaked in ethanol or cleaned with a strong, corrosive solution.
- Clean both ends of the light guide if the light seems to have lost its brightness.

3. Specifications

Model : CP5-O <with Fiber Optic>
ISO 9168 Type B Coupling

Development and Manufacturing
J. MORITA MFG. CORP.
860 Higashihama Minami-cho,
Fushimi-ku, Kyoto 612-8533, Japan
T +81. (0)75. 611 2141; F +81. (0)75. 622 4595
Morita Global Website www.morita.com

Distribution
J. Morita Corporation
3-33-18 Tarumi-cho, Suita-shi, Osaka 564-8650, Japan
T +81. (0)6. 6380 1521; F +81. (0)6. 6380 0585

J. Morita USA, Inc.
9 Mason, Irvine CA 92618, USA
T +1. 949. 581 9600; F +1. 949. 581 8811

J. Morita Europe GmbH
Justus-von-Liebig-Strasse 27a, 63128 Dietzenbach, Germany
T +49. (0)6074. 836 0; F +49. (0)6074. 836 299

Morita Dental Asia Pte. Ltd.
3 Science Park Drive,
#01-05 The Franklin Singapore Science Park1, Singapore 118223
T +65. 6779. 4795; F +65. 6777. 2279

J. Morita Corporation Australia & New Zealand
Suite 2.05, 247 Coward Street, Mascot NSW 2200, Australia
T +61. (0)2. 9667 3555; F +61. (0)2. 9667 3577

EU Authorized Representative under the European Directive 93/42/EEC
MEDICAL TECHNOLOGY PROMEDY CONSULTING GmbH
Altenhofstraße 80, 66386 St. Ingbert, Germany T +49. 6894 581020; F +49. 6894 581021
The authority granted to the authorized representative, MEDICAL TECHNOLOGY PROMEDY CONSULTING GmbH, by J. MORITA MFG. CORP. is solely limited to the work of the authorized representative with the requirements of the European Directive 93/42/EEC for product registration and incident report.

J. Morita Corporation Middle East
4 Tag Al Roasaa, Apartment 902, Saba Pacha 21311 Alexandria, Egypt
T +20. (0)3. 58 222 94; F +20. (0)3. 58 222 96

J. Morita Corporation India
Felix Office No. 908, L. B. S. Marg, Opp. Asian Paints,
Bhandup (West), Mumbai 400078, India
T +91-22-2595-3482

J. MORITA MFG. CORP. Indonesia Representative Office
28F, DBS Bank Tower, Jl. Prof. Dr. Satrio Kav. 3-5, Jakarta 12940, Indonesia
T +62-21-2988-8332; F +62-21-2988-8201

Siamdent Co., Ltd.
711/10 Mu 5, Thakham, Bangkokkong, Chachuengsao 24130, Thailand
T +66. 38. 573042; F +66. 38. 573043 www.siamdent.com