



Lubrina Fast, Economical Handpiece Maintenance



J.MORITA MFG.CORP.

Lubrina features at a glance

- **NEW!** More powerful with increased air flow speed
- **NEW!** Compatible with 4-hole, 5-hole & 6-pin
- Compatible with multiple brands of spray lubricant
- Operates with up to two different sprays per cycle
- Accommodates wide variety of handpieces & attachments
- Services up to four handpieces in 100 seconds

Thinking ahead. Focused on life.



Newly Updated - More Powerful & Versatile

Lubrina cleans and purges handpieces, and a variety of other equipment, via a rapid air blow system. The speed of air flow has been substantially increased resulting in more power. It delivers superior performance in the same amount of time – just 25 seconds per handpiece. Lubrina is currently the ONLY system available that accepts multiple brands of spray lubricant including the use of two different sprays in the same cycle. And now, also compatible with 5-hole connections, it's more versatile than ever before.

Services four handpieces in 100 seconds

Requiring approximately 25 seconds per handpiece, Lubrina can service up to four handpieces in 100 seconds – including lubrication and purging. Compared with manual lubrication, Lubrina uses only 1/3 the lubricant.

Built-in chuck maintenance system

Lubrina is the first handpiece maintenance system with a built-in chuck maintenance system (only for Morita instruments).

Maintenance for a variety of instruments

Lubrina offers you fully automatic, highly efficient, user-friendly handpiece maintenance for contra-angles, straight handpieces, air driven handpieces, and air scalars. By replacing the coupling, Lubrina facilitates the maintenance of Tri Auto ZX and Root ZX® II contra-angles, air driven handpieces (ISO 9168 / Type 3 joint*) as well as contra-angles and straight handpieces from other manufacturers (ISO 3964 / E-Type joint).

To order or for more information, call 888-JMORITA (566-7482) or contact your dealer.



Unique dual lubrication system

Featuring a one-of-a-kind dual lubrication system, Lubrina enables lubrication of two instruments with one spray – or, if desired, the use of two different sprays. Lubrina does not require a special spray. It can be operated with any authorized brand including those from other manufacturers by using an optional spray can stand.

Air suction system

The air suction system eliminates lubricant mist, so there is less odor. Excess lubricant goes to the lubricant absorber through a return circuit to keep the operating area clean.

Lubricant indicator

Lubrina is equipped with a lubrication sensor to monitor the system status. The sensor will detect an empty spray condition and will illuminate an indicator light.

Selectable lubrication time option

Lubrina allows you to select between two lubrication time modes: short mode and long mode.

*Lubrina is compatible with the following connections.



6-Pin Connection



4-Hole Connection



5-Hole Connection



Lubrina services up to 4 handpieces simultaneously and is compatible with a wide variety of instruments, including instruments from other manufacturers.



The unique dual lubrication system

Lubrina Specifications

Name	Lubrina
Model	HIM-1
Power requirement	AC 100 – 240 V; 50/60 Hz
Power consumption	18 VA
Operating air pressure	0.3 - 0.5 MPa / 43.5 - 72.5 psi
Recommended air pressure	0.35 MPa / 50.8 psi
Air flow	40 – 60 NL/min. (1.41 - 2.12 CFM)
Weight	approx. 22 lb. (10 kg)
Dimensions (W x H x D)	12" x 15" x 12" (300 x 370 x 300 mm)

Ordering Information

Lubrina Handpiece	
Maintenance System	24-5343054
System Contains:	Oil Absorption Pads (5), Oil Absorption Case, Power Cord, Tray Filter, Tray, AR Spray Can Stand, Air Tube, AR Spray (420 ml)

Other Accessories/Replacement Parts:

Alpha Coupling	24-6812600	Spray Can Stand for Morita	
TR (ISO 3964) Coupling	24-6812602	AR Spray (420 ml)	24-6812610
TwinPower (4H) Coupling	24-6812604	Spray Can Stand for KaVo®	
Root ZX II Coupling	24-6812606	Spray (500 ml)	24-6812622
Tri Auto ZX Coupling	24-6812608	Spray Can Stand for Bien-Air®	
KaVo® Coupling	24-6812616	Lubrifiuid Spray (500 ml)	24-6812624
Sirona® Coupling	24-6812618	Spray Can Stand for W&H®	
W&H® Roto Quick Coupling	24-6812619	Service Oil F1 Spray (400 ml)	24-6812626
AR Spray (420 ml)	24-5010212	Spray Can Stand for Sirona®	
Lubrina Oil Absorption Pad (5)	24-6812612	T1 Spray (250 ml)	24-6812628
Lubrina Oil Absorption Case	24-6812631	Spray Can Stand for NSK®	
Lubrina Tray	24-6812632	Pana Spray (480 ml)	24-7373414
Lubrina Tray Filter	24-6812614		
Lubrina Air Tube	24-5890917		
Lubrina Power Cord	24-6812657		