

TRI AUTO ZX MAINTENANCE INFORMATION

Use the following instructions to clean, lubricate with AR Oil, and sterilize the Tri Auto ZX.

For detailed instructions for use on this product, please contact J. Morita USA at 877-JMORITA (566-7482).

CLEANING, LUBRICATING AND STERILIZING THE CONTRA HEAD

1. Lubrication

* Lubricate the contra head once a month.


- a. Place the contra head in a paper cup with the connection end facing up.
- b. Put ten (10) drops of AR oil on the gear and wait for ten (10) minutes.

NOTE:

- Leave the contra head in a paper cup for at least ten (10) minutes so that the oil is thoroughly absorbed by the contra head mechanism.
- c. Take the contra head out of the paper cup and wipe off any excess oil which may have seeped out.




CODE No.7503800

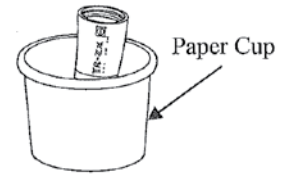

2. Autoclave Sterilization

* It is not necessary to lubricate the contra head before autoclaving it.

- a. Wipe the contra head with a piece of gauze and disinfecting alcohol.
- b. Place the contra head into a sterilization pouch. Autoclave it according to the autoclave manufacturer's instructions for sterilizing small equipment.
- c. Autoclave the contra head for at least 6 minutes at 135°C (275°F). Drying temperature must not exceed 135°C (275°F).
- d. Remove the contra head from the sterilization pouch as soon as it has been sterilized and attach it to the cordless handpiece.

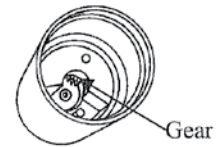

NOTE:

- Do not wipe the contra head or cordless handpiece with solutions containing **formalin cresol** or **hypochlorite**. These chemicals will damage plastic components and also corrode the battery and circuit boards inside the cordless handpiece. Immediately wipe these chemicals off if they get on the instrument.
- Do use a disposable plastic sleeve to protect the cordless handpiece from any solutions.

J. Morita USA, Inc.
9 Mason
Irvine, CA 92618
www.jmoritausa.com